


CLEANCUT®

The Step-In Tub Experts



PRODUCT CATALOG

Table of Contents

- 
- 4 **CleanCut Advantages**
 - 5 **Product Comparison**
 - 6 **The CleanCut Step®, Deep Step® & Ultra-Low®**
 - 7 **The CleanCut Step®**
 - 8 **The CleanCut Deep Step®**
 - 9 **The CleanCut Ultra-Low®**
 - 10 **The CleanCut Convertible®**
 - 12 **Ordering & Shipping**
 - 13 **FAQs**
 - 14 **Accessories**
 - 15 **Sizing Guides**

VISIT

cleancutbath.com

CALL

877-882-7837

EMAIL

info@cleancutbath.com

CleanCut Advantages

CleanCut Installer Benefits

- Convert leads for customers looking for a lower price point
- Pairs well with other safety/accessibility products
- Help customers regain & maintain independence and safety in their homes
- Tap into the rapidly growing aging-in-place customer segment
- Quickly installed with high profit margins
- Exclusive rates and bulk discounts for your company
- Full marketing resource materials (print materials, professional photos, logos, and videos for your company's use)
- Step by step installation videos available (scan the How To QR code on the right)
- Receive a complete kit every order – includes product, two-part support box structure, industrial-grade adhesive with application nozzle, cutting template and installation guide



MORE
INFO
VIDEO



HOW-TO
INSTALL
VIDEO

Product Advantages

- Molded-in non-skid treadplate on all products
- Multiple product and adhesive color options
- Units made of strong, durable polyethylene material
- Full kit warranty, including adhesive
- Free shipping on all kit orders

MADE
IN THE
USA

Product Comparison

Each of our products has unique benefits.
Here's how they stack up...


	THE CLEAN CUT STEP	THE CLEAN CUT DEEP STEP	THE CLEAN CUT ULTRA-LOW	THE CLEAN CUT CONVERTIBLE
Installs on existing tub	✓	✓	✓	✓
Molded-in non-skid treadplate	✓	✓	✓	✓
Installs in less than one day	✓	✓	✓	✓
Installs on all types of tubs including: Fiberglass, Acrylic, Steel and Cast Iron	✓	✓	✓	✓
Kit includes internal support structure	✓	✓	✓	✓
Kit includes waterproof membrane				✓
Allows for full bath				✓
Optimal for deep tubs		✓	✓	
Ideal for jacuzzi, curved or garden style tubs		✓ Extra-Wide size		
Available in white & beige	✓	✓	✓	✓
Template height	8.5"	11"	14"	8.5"
Size options	Narrow Medium Wide	Wide Extra-Wide	Wide	Wide

THE **CLEANCUT**
STEP

THE **CLEANCUT**
DEEP STEP

THE **CLEANCUT**
ULTRA-LOW

Convert Tub to Step-In Shower

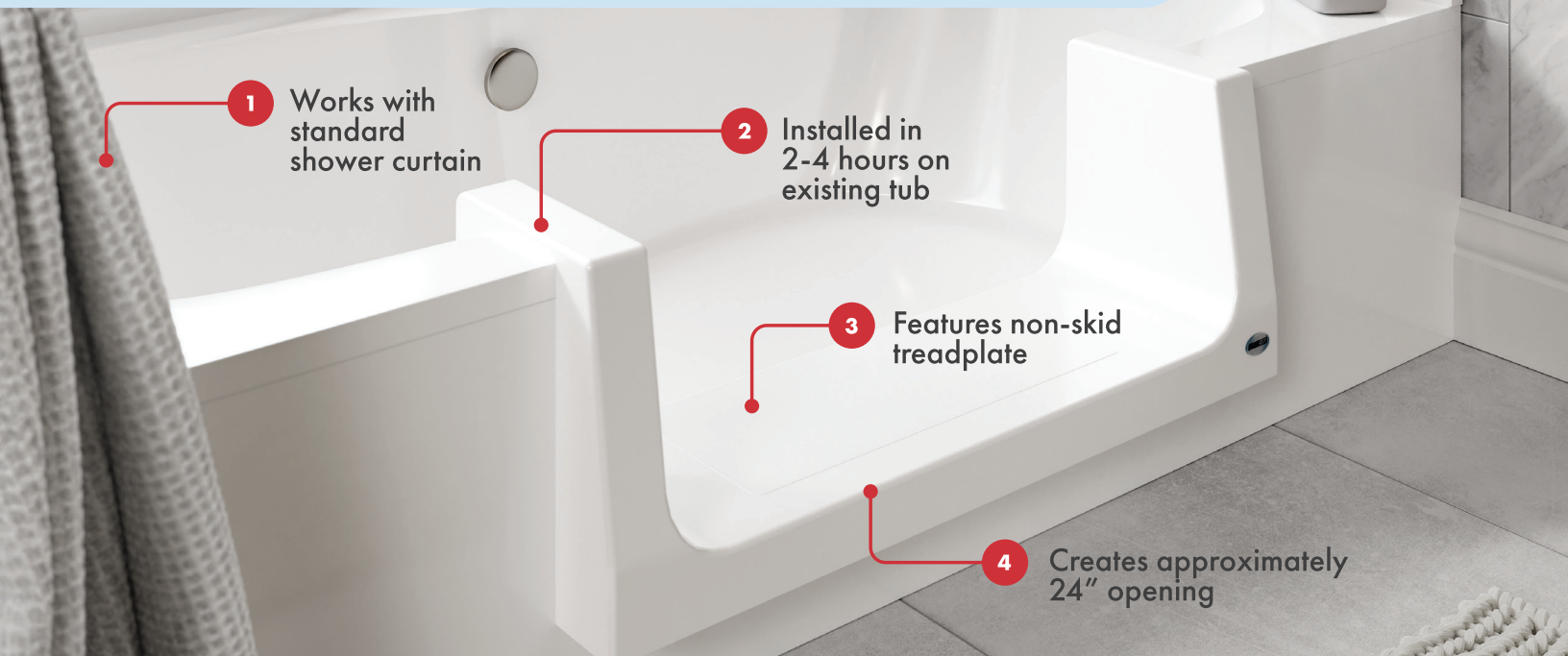

5 Available in 2 colors

WHITE

BEIGE



6 EACH KIT INCLUDES:

One product, one 2-part Support Structure, two cartridges of Industrial Grade Adhesive with Application Nozzle, one Cutting Template, and Installation Guide.



7 Step, Deep Step and Ultra-Low all share the listed qualities above, but each feature differing heights and widths to accommodate different tub shapes and depths.

THE CLEAN CUT[®] STEP



HOW-TO
INSTALL
VIDEO

Amount of original step-in height removed	8.5"
Color options	White • Beige
Size options	Narrow • Medium • Wide

Reference sizing information on these pages: Narrow **16** Medium **17** Wide **18**



The project was given to our handyman. He viewed the videos on the installation procedures and went right to it. Measure twice and cut once. No mistakes and was not difficult at all. The seniors at our facility appreciated the inserts. Good product. – Rudy S. *Cleveland, OH*

THE **CLEANCUT**

DEEP STEP



SCAN FOR A
HOW-TO VIDEO
ON SIZING



Amount of original step-in height removed	11"
Color options	White • Beige
Size options	Wide • Extra Wide

Extra-wide width designed to accommodate curved, garden, jetted and other unique types of tubs

Reference sizing information on these pages: Wide **19** Extra Wide **20**



Really a wonderful product we installed in my aging mother's bathroom and feel that this is such a safer option for her. Would recommend. – Sherri S. El Paso, IL

THE **CLEANCUT**

ULTRA-LOW[®]



SCAN FOR A
HOW-TO VIDEO
ON SIZING



14.75" or
taller on inside
of tub for
Ultra-Low
to fit

Amount of original step-in height removed	14"
Color options	White • Beige
Size options	Wide

Reference sizing information on this page: Wide **21**



This product was super easy to install in about 2hrs for our first one and our customer loves it. What a wonderful product to have and to offer to our customers as an alternative to remodeling the entire bathtub. – Daniel K. *Wautoma, WI*

THE **CLEANCUT**

CONVERTIBLE



Effortless Transition From Step-In Shower to Full Bath



HOW-TO
INSTALL
VIDEO

<i>Amount of original step-in height removed</i>	8.5"
<i>Color options</i>	White • Beige
<i>Size options</i>	Wide

Reference sizing information on this page: Wide **22**




Works great installed for customer with Parkinson's. Made their tub accessible.
Would highly recommend for alternative to a walk-in shower. – Cory F. Estherville, IA

Ordering & Shipping

When you are ready to order, please login to your account and order online at www.cleancutbath.com/shop. Upon account creation, please login to place your order. This is the fastest and most accurate way to ensure that you receive your order in a timely manner. If your order is placed before 2pm EST, it will ship the same day. You will receive an email with your tracking information once it has been tendered to FedEx.

**FREE
SHIPPING**

FedEx Ground • Palletize option for 24+ kits

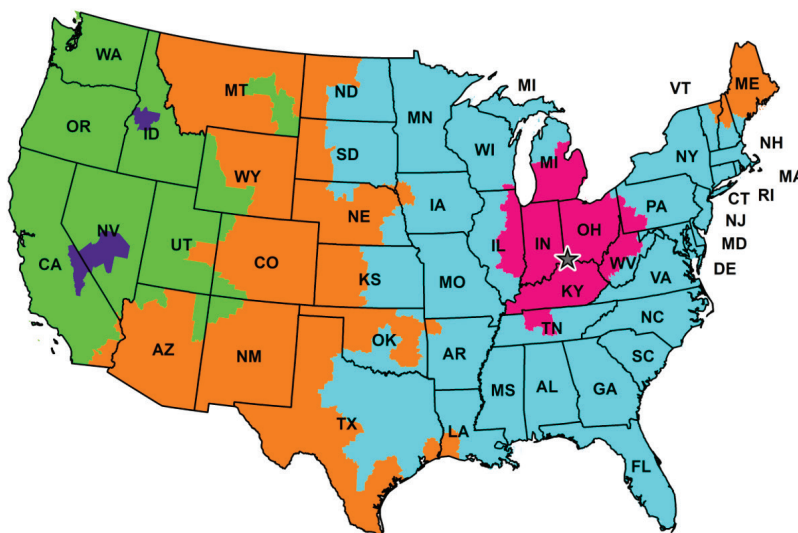

When will your FedEx Ground shipment be scheduled for delivery?

	1 Day	2 Days	3 Days	4 Days	5 Days	6 Days	7+ Days
If picked up on...	Will be delivered by...						
Monday	Tue	Wed	Thu	Fri	Mon@	Tue@	Wed ⁼
Tuesday	Wed	Thu	Fri	Mon	Tue@	Wed@	Thu ⁼
Wednesday	Thu	Fri	Mon	Tue	Wed@	Thu@	Fri ⁼
Thursday	Fri	Mon	Tue	Wed	Thu@	Fri@	Mon ⁼
Friday	Mon	Tue	Wed	Thu	Fri@	Mon@	Tue ⁼
Saturday	Tue	Wed	Thu	Fri	Mon@	Tue@	Wed ⁼
Sunday	Tue	Wed	Thu	Fri	Mon@	Tue@	Wed ⁼

@ The following week
⁼ Earliest possible delivery the following week
 © 2024 FedEx. All rights reserved.

This map illustrates service schedules in business days as of April 2024 for FedEx Ground Shipments



Great product! Fast shipping and a fair price. Easy ordering online or on the phone, very knowledgeable and helpful customer service. I am very glad I found this company, it's been a great addition to my business! – Rob S. *Detroit Lakes, MN*

1 Type of tubs able to be installed

CleanCut products can be installed on any type of tub, including fiberglass, acrylic, steel and cast-iron.

2 Tubs with jets

CleanCut products can possibly be installed on tubs with jets, depending on the tub shape. Jets would need to be capped off prior to installation and will not be able to be used. Keep in mind where the plumbing is located on the tub as well, as you do not want to cut into the plumbing when cutting the sidewall of the tub.

3 Tubs with liners

We do not recommend installing a CleanCut product on a tub with a liner, as you might not know what is under the liner. If it is a cast iron tub, the heat from the cut could damage the liner. Also, the liner could become loose from the vibration of the cutting during installation.

4 Clawfoot tubs

Unfortunately, CleanCut products cannot be installed on clawfoot tubs due to the inability to install the support structure underneath the product to the floor.

5 Curved and Garden Tubs

With curved and garden tubs, it really depends on the specific tub. We do have the Deep Step in an Extra-Wide size to accommodate many curved and garden tubs. If you have a customer with a curved or garden tub, feel free to send a photo of the tub to info@cleancutbath.com and our customer service team can take a closer look.

6 Weight limit

CleanCut products have a weight limit of 300 lbs. when installed properly.

7 What is product made of?

All CleanCut inserts are made of composite polyethylene plastic – an extremely durable, hard plastic.

8 What other tools/materials are recommended for installation?

Butane Torch	Ear Protection	Pressure Treated 2x4's
Cardstock Paper	Extension Cord	Protective Eye Wear
Caulking Gun	Fire Extinguisher	Reciprocating Saw with Metal Blade
Chisels	Gloves	Scissors
Deburring Tool	Hammer	Screwdriver
(i.e. drywall plane, rasp)	Isopropyl Alcohol	Screws - Assorted Sizes
Drain Screens	Mirror (small)	Sharpie Marker (colored)
Drill & Drill Bits	Paper Towels	Side Grinder with 4.5" Diamond Blade
Drop Cloths	Jigsaw with no more than 1.75" blade	Utility Knife
Dust Masks	Masking Tape	Vacuum with Attachment
Dustpan & Brush	Pliers	Wood Shims

Accessories

Cut Buddie 5" Grinder Shroud

The CutBuddie is ideal for dust control when using a hand grinder to cut the tub for a CleanCut product. Made of durable polycarbonate. Clear shroud gives installer visibility needed for optimal control. CutBuddie connects with vacuum for easy dust collection.



Diteq Diamond Brazed Segmented Blade

Recommended blade for use with grinder during CleanCut installation (particularly cast iron). Fits any grinder with a 5/8" or 7/8" arbor.

Diameter & Width: 4.5" x .095" x 7/8"-5/8



16.5" Tub Bar

16.5" tub bar provides a comfortable, safe grip for stepping into and out of the tub. Handle is perpendicular to tub. Product width adjusts from 3" to 6.25" and is an easy, tool-free installation. Perfectly paired with a CleanCut bath accessibility product. Not suitable for fiberglass or acrylic tubs.



Love it. Quality construction made in USA. A perfect alternative to a very expensive full shower/tub remodel. I have exactly what I need and wanted. No more worrying about stepping over the edge of the tub and risk falling. Thank you CleanCut! – Judy C. Hamilton, MT



SIZING GUIDES




SCAN FOR A HOW-TO VIDEO ON SIZING & MEASURING



SIZING & MEASUREMENT GUIDE FOR NARROW SIZE

TUB REQUIREMENTS:

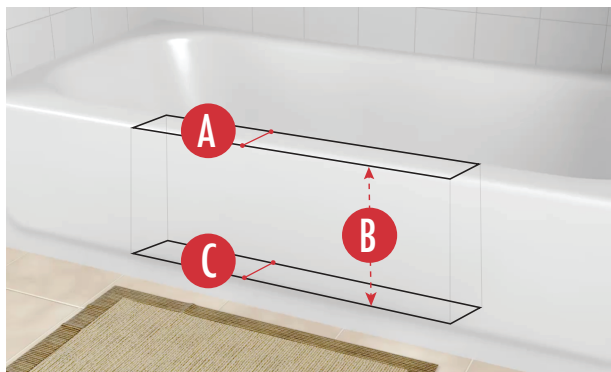

Max * Tub-Wall Width at Top: 4"

Bottom Width Measure Depth: 9.75"


Distance to measure down from top of tub to establish "Max Tub-Wall Width at Bottom"

Max Tub-Wall Width at Bottom: 4.75"

Note: Bottom tub width is typically wider than top width.

***IF YOUR TUB EXCEEDS THESE MAX SIZE MEASUREMENTS, THIS PRODUCT WILL NOT WORK WITH YOUR TUB.**

TEMPLATE (CUT LINES) AND PRODUCT OVERLAP MEASUREMENTS:

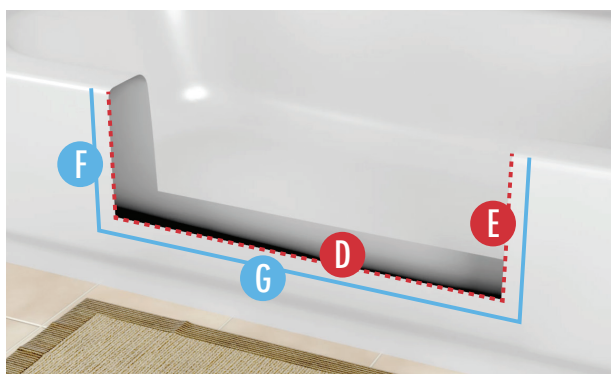

Template Length: 25.5"


Template Height: 8.5"

AMOUNT PRODUCT OVERLAPS PAST CUT LINES:

Left and Right Side Overlap: 1.25"

Bottom Overlap: 1.25"

ACTUAL PRODUCT MEASUREMENTS:

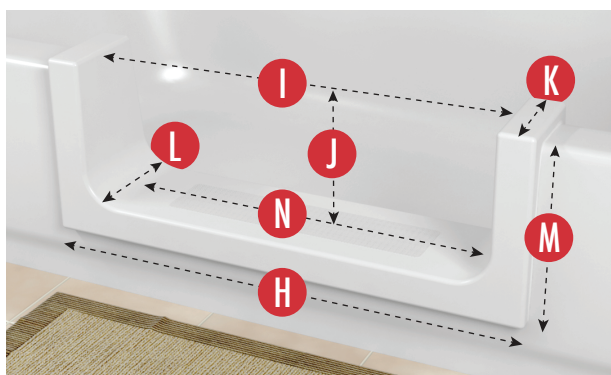

Product Length: 28"

Opening Left to Right at Top: 24"

Product Opening Top to Bottom: 8.5"

Product Width at Top: 5"

Product Width at Bottom: 5.75"


Product Height: 10.25"

Opening Left to Right at Bottom: 20.75"



SIZING & MEASUREMENT GUIDE FOR MEDIUM SIZE

TUB REQUIREMENTS:

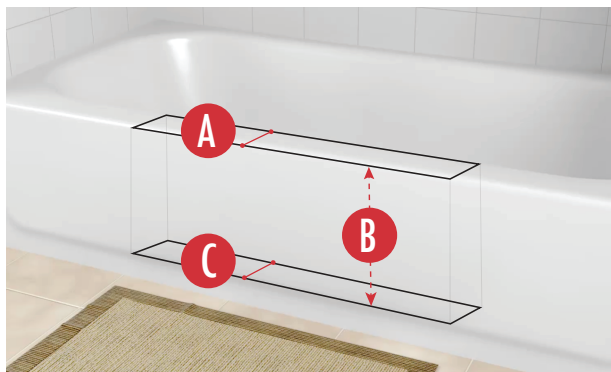

Max * Tub-Wall Width at Top: 4.5"

Bottom Width Measure Depth: 9.75"


Distance to measure down from top of tub to establish "Max Tub-Wall Width at Bottom"

Max Tub-Wall Width at Bottom: 5.75"

Note: Bottom tub width is typically wider than top width.

***IF YOUR TUB EXCEEDS THESE MAX SIZE MEASUREMENTS, THIS PRODUCT WILL NOT WORK WITH YOUR TUB.**

TEMPLATE (CUT LINES) AND PRODUCT OVERLAP MEASUREMENTS:

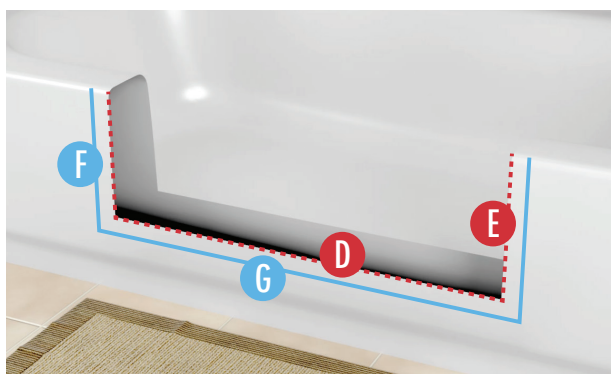

Template Length: 25.5"


Template Height: 8.5"

AMOUNT PRODUCT OVERLAPS PAST CUT LINES:

Left and Right Side Overlap: 1.25"

Bottom Overlap: 1.25"

ACTUAL PRODUCT MEASUREMENTS:

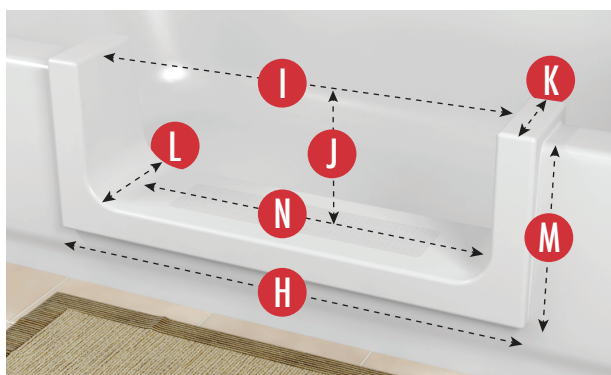

Product Length: 28"

Opening Left to Right at Top: 24"

Product Opening Top to Bottom: 8.25"

Product Width at Top: 5.5"

Product Width at Bottom: 6.75"


Product Height: 10.25"

Opening Left to Right at Bottom: 20.75"



SIZING & MEASUREMENT GUIDE FOR WIDE SIZE

TUB REQUIREMENTS:

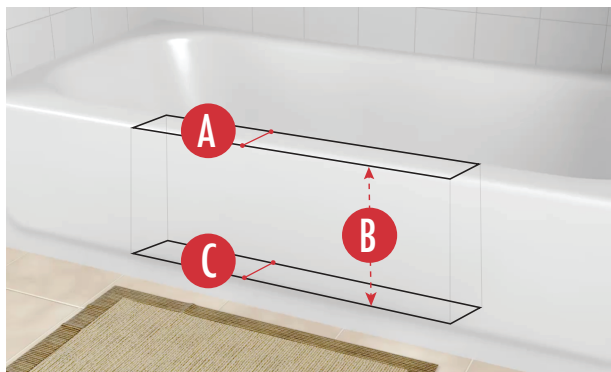

Max * Tub-Wall Width at Top: 6"

Bottom Width Measure Depth: 9.75"


Distance to measure down from top of tub to establish "Max Tub-Wall Width at Bottom"

Max Tub-Wall Width at Bottom: 7.25"

Note: Bottom tub width is typically wider than top width.

***IF YOUR TUB EXCEEDS THESE MAX SIZE MEASUREMENTS, THIS PRODUCT WILL NOT WORK WITH YOUR TUB.**

TEMPLATE (CUT LINES) AND PRODUCT OVERLAP MEASUREMENTS:

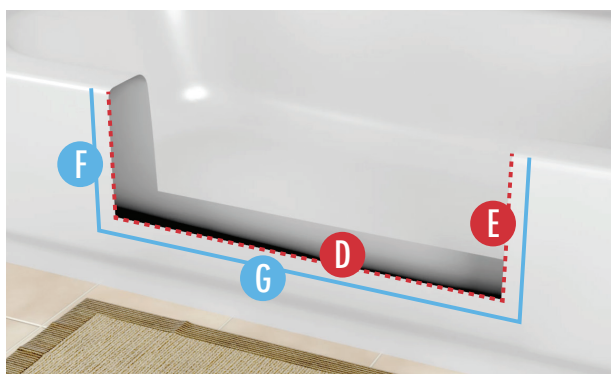

Template Length: 25.5"


Template Height: 8.5"

AMOUNT PRODUCT OVERLAPS PAST CUT LINES:

Left and Right Side Overlap: 1.25"

Bottom Overlap: 1.25"

ACTUAL PRODUCT MEASUREMENTS:

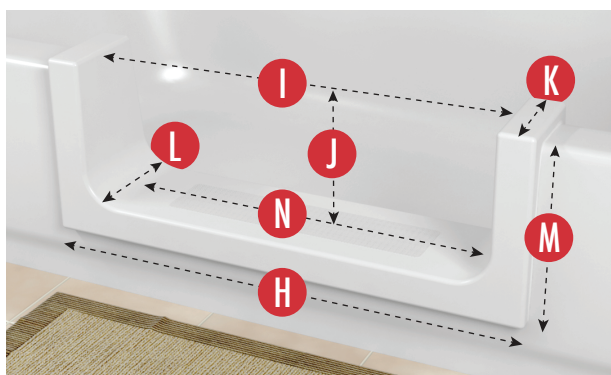

Product Length: 28"

Opening Left to Right at Top: 24"

Product Opening Top to Bottom: 8.5"

Product Width at Top: 7"

Product Width at Bottom: 8.25"


Product Height: 10.25"

Opening Left to Right at Bottom: 20.75"

DEEP STEP

SIZING & MEASUREMENT GUIDE FOR WIDE SIZE

TUB REQUIREMENTS:

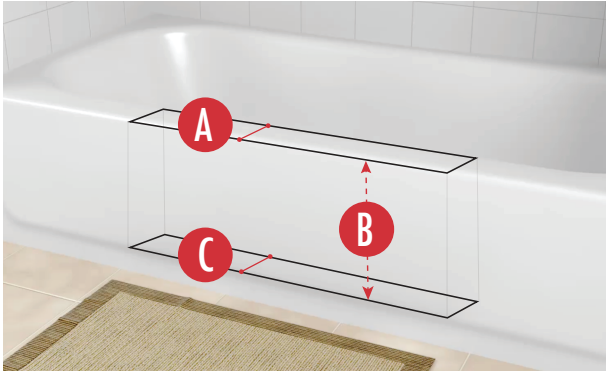

Max * Tub-Wall Width at Top: 6"

Bottom Width Measure Depth: 11.75"


Distance to measure down from top of tub to establish "Max Tub-Wall Width at Bottom"

Max Tub-Wall Width at Bottom: 7.25"

Note: Bottom tub width is typically wider than top width.

***IF YOUR TUB EXCEEDS THESE MAX SIZE MEASUREMENTS, THIS PRODUCT WILL NOT WORK WITH YOUR TUB.**

TEMPLATE (CUT LINES) AND PRODUCT OVERLAP MEASUREMENTS:

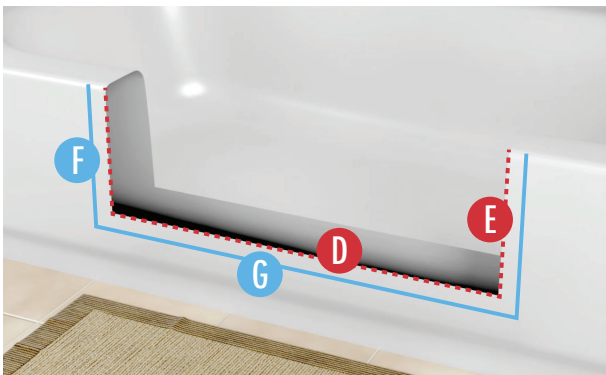

Template Length: 25.5"


Template Height: 11"

AMOUNT PRODUCT OVERLAPS PAST CUT LINES:

Left and Right Side Overlap: 1.25"

Bottom Overlap: 1"

ACTUAL PRODUCT MEASUREMENTS:

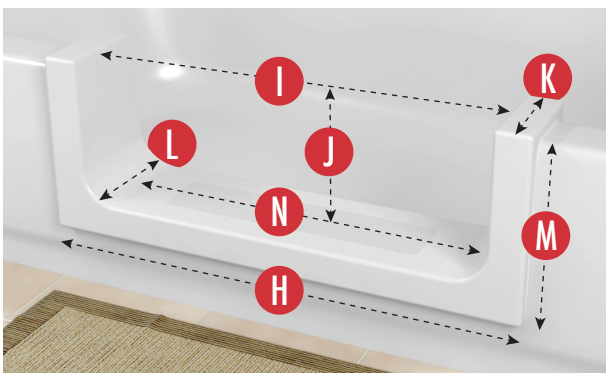

Product Length: 28"

Opening Left to Right at Top: 24"

Product Opening Top to Bottom: 11"

Product Width at Top: 7"

Product Width at Bottom: 8.25"


Product Height: 12.75"

Opening Left to Right at Bottom: 20"

DEEP STEP

SIZING & MEASUREMENT GUIDE FOR EXTRA WIDE SIZE

TUB REQUIREMENTS:

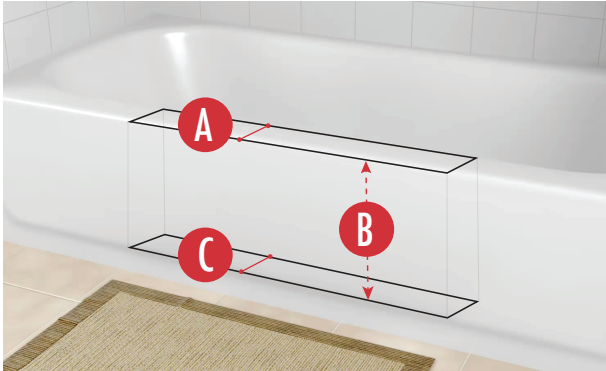

Max * Tub-Wall Width at Top: 8"

Bottom Width Measure Depth: 11.75"


Distance to measure down from top of tub to establish "Max Tub-Wall Width at Bottom"

Max Tub-Wall Width at Bottom: 9"

Note: Bottom tub width is typically wider than top width.

***IF YOUR TUB EXCEEDS THESE MAX SIZE MEASUREMENTS, THIS PRODUCT WILL NOT WORK WITH YOUR TUB.**

TEMPLATE (CUT LINES) AND PRODUCT OVERLAP MEASUREMENTS:

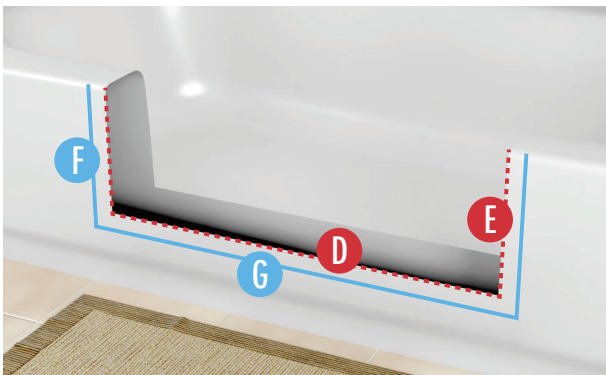

Template Length: 25.5"


Template Height: 11"

AMOUNT PRODUCT OVERLAPS PAST CUT LINES:

Left and Right Side Overlap: 1.25"

Bottom Overlap: 1"

ACTUAL PRODUCT MEASUREMENTS:

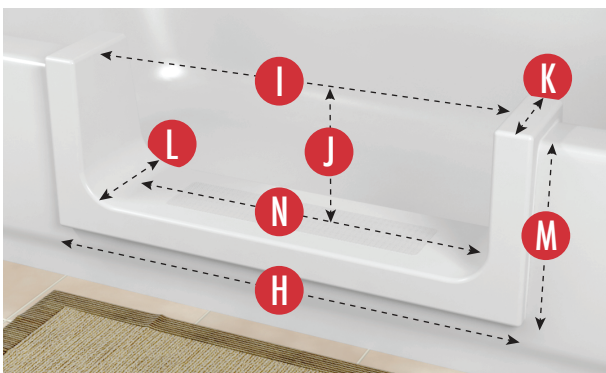

Product Length: 28"

Opening Left to Right at Top: 24"

Product Opening Top to Bottom: 11"

Product Width at Top: 9"

Product Width at Bottom: 10"

Product Height: 12.75"

Opening Left to Right at Bottom: 20"


THE **CLEANCUT**

ULTRA-LOW

SIZING & MEASUREMENT GUIDE

The Ultra-Low is designed to fit high-profile tubs. If the tub is less than 14.75 inches tall on the outside & inside of tub, please take a look at the Step or Deep Step sizing guides.

TUB REQUIREMENTS:

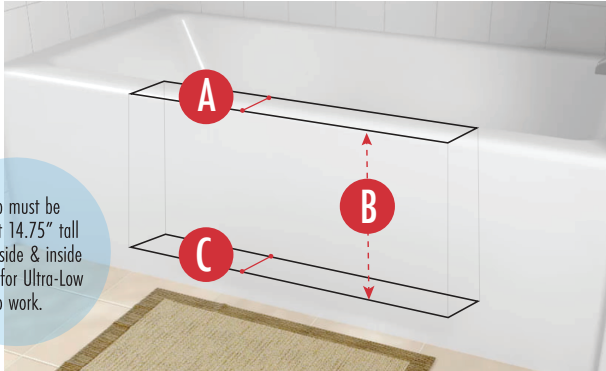

Max * Tub-Wall Width at Top: 6"

Bottom Width Measure Depth: 14.75"


Distance to measure down from top of tub to establish "Max Tub-Wall Width at Bottom"

Max Tub-Wall Width at Bottom: 7.25"

Note: Bottom tub width is typically wider than top width.

***IF YOUR TUB EXCEEDS THESE MAX SIZE MEASUREMENTS, THIS PRODUCT WILL NOT WORK WITH YOUR TUB.**

TEMPLATE (CUT LINES) AND PRODUCT OVERLAP MEASUREMENTS:

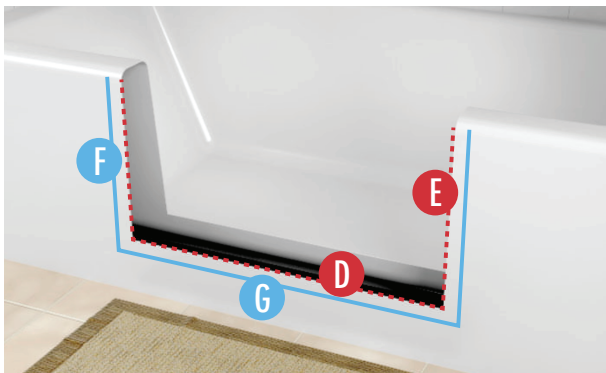

Template Length: 25.5"


Template Height: 14"

AMOUNT PRODUCT OVERLAPS PAST CUT LINES:

Left and Right Side Overlap: 1.5"

Bottom Overlap: 0.50"

ACTUAL PRODUCT MEASUREMENTS:

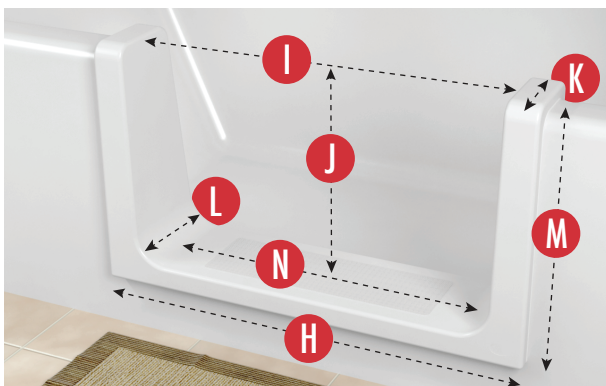

Product Length: 28"

Opening Left to Right at Top: 24.25"

Product Opening Top to Bottom: 13.75"

Product Width at Top: 7"

Product Width at Bottom: 8.25"


Product Height: 15.75"

Opening Left to Right at Bottom: 20.75"

CONVERTIBLE

SIZING & MEASUREMENT GUIDE

TUB REQUIREMENTS:

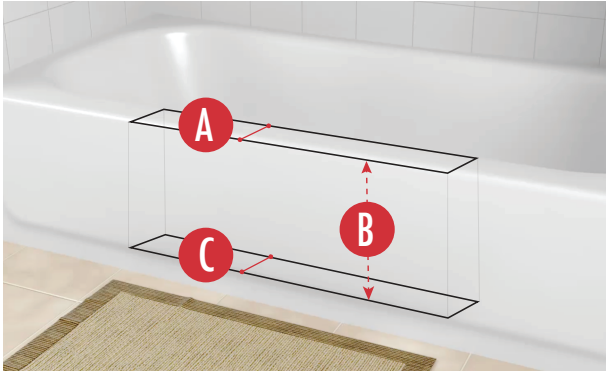

Max * Tub-Wall Width at Top: 6"

Bottom Width Measure Depth: 10.5"


Distance to measure down from top of tub to establish "Max Tub-Wall Width at Bottom"

Max Tub-Wall Width at Bottom: 7"

Note: Bottom tub width is typically wider than top width.

***IF YOUR TUB EXCEEDS THESE MAX SIZE MEASUREMENTS, THIS PRODUCT WILL NOT WORK WITH YOUR TUB.**

TEMPLATE (CUT LINES) AND PRODUCT OVERLAP MEASUREMENTS:

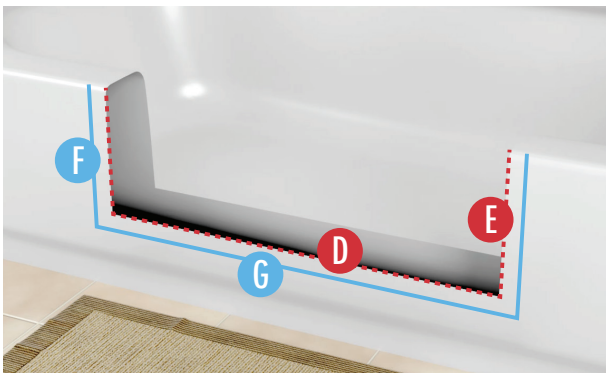

Template Length: 25.5"


Template Height: 8.5"

AMOUNT PRODUCT OVERLAPS PAST CUT LINES:

Left and Right Side Overlap: 1.75"

Bottom Overlap: 1.5"

ACTUAL PRODUCT MEASUREMENTS: (PLUG INSERT NOT SHOWN)

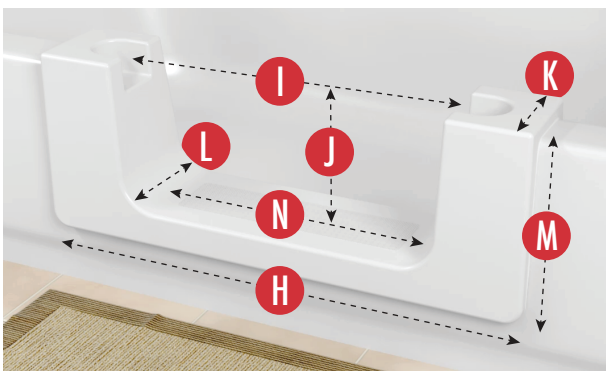

Product Length: 28.75"

Opening Left to Right at Top: 18.75"

Product Opening Top to Bottom: 8.5"

Product Width at Top: 7"

Product Width at Bottom: 8"

Product Height: 10.75"

Opening Left to Right at Bottom: 15.5"

Limited Warranty

Subject to the terms of this Limited Warranty, Safeway Safety Step, LLC (DBA "CleanCut") will, at its election in its sole discretion, repair or replace the CleanCut Step, CleanCut Ultra- Low, CleanCut Door, or CleanCut Convertible ("Product"), or issue a Product refund, if the Product is found by CleanCut to be defective in material or workmanship (or both) during the relevant period set forth below.

Limited Warranty Period

The warranty period begins on the date that the Product is shipped to the original purchaser and continues for one (1) year so long as the Product is used by the original purchaser for personal, family or household purposes. This warranty is not transferrable and does not apply to subsequent owners, transferees or users.

This limited warranty DOES NOT COVER the following:

Defect or damage arising from shipping, installation, alterations or modifications of any type, accidents, abuse, misuse, lack of proper maintenance (including failures to maintain caulking) and cleaning, whether caused by an installer, plumbing contractor, service company, the original purchaser or any other person.

ANY DEFECTS RESULTING FROM IMPROPER INSTALLATION OR MAINTENANCE OF THE PRODUCT, AND IN NO EVENT SHALL THIS LIMITED WARRANTY APPLY IF THE PRODUCT IS NOT INSTALLED BY A LICENSED CONTRACTOR OR OTHER TRAINED PROFESSIONAL.

Deterioration through normal wear and tear and the expense of normal maintenance.

Shipping cost for returning products for repairs or replacement under the limited warranty and labor or other cost incurred in connection with removal or installation under this limited warranty.

ANY LIABILITY FOR CONSEQUENTIAL OR INCIDENTAL DAMAGES, ALL OF WHICH ARE HEREBY EXPRESSLY DISCLAIMED, OR EXTENSIONS BEYOND THE DURATION OF THIS LIMITED WARRANTY OF ANY IMPLIED WARRANTIES, INCLUDING THOSE OF MERCHANTABILITY OR FITNESS FOR AN INTENDED PURPOSE.

Depending on your state, some of these limitations may not apply to you.

Responsibility for compliance with local code requirements. This warranty shall become effective upon receipt by CleanCut of warranty registration. To register warranty, please visit www.cleancutbath.com/warranty and note the Reference Number provided at the top of this form. For service under this warranty, you must notify CleanCut in writing within ten (10) days of discovery of a defect at: 5242 Rialto Road, West Chester, OH 45069 or by email at info@cleancutbath.com. All such notices must include the date when the defect was discovered, a description of the defect, the name and address of the installer of the Product and evidence of the date of purchase and date of delivery. Please allow a reasonable time for commencement and completion of warranty repairs. No other company or person has any authority to make any warranties or representations on behalf of CleanCut for the Product. This limited warranty does not affect the other statutory rights you may have, which vary from state to state.



VISIT

cleancutbath.com

CALL

877-882-7837

EMAIL

info@cleancutbath.com